


Osteoarthritis (OA) / Rheumatoid Arthritis (RA)

Bilateral Symptoms:

Place one electrode above lateral joint line proximal to lower head of femur is slight depression (lat. cutaneous nerve of the thigh); place other electrode at medial joint line between semimembranosus and semitendinosus tendons (saphenous nerve).

KNEE Bilateral Symptoms

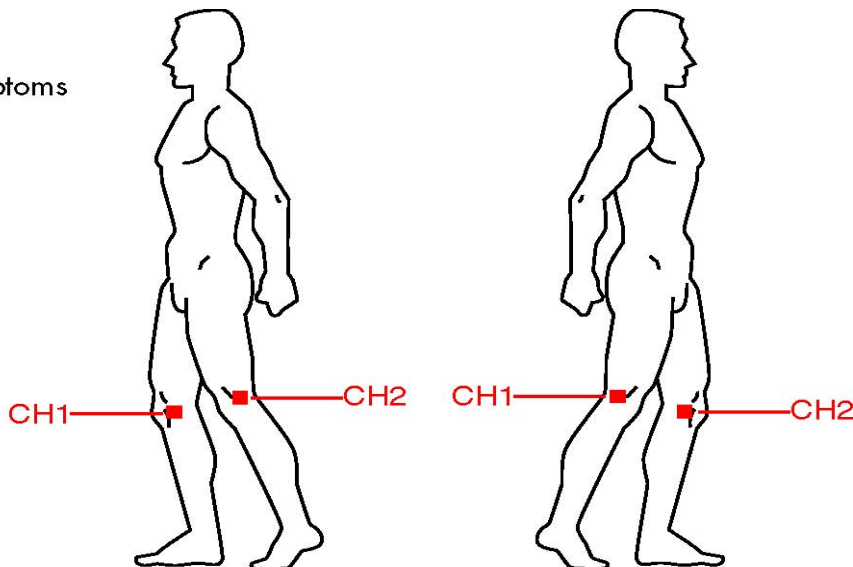

Unilateral Symptoms:

Place one channel at B54/40 (center of the popliteal crease) and "Hetting" (superior to patella at midline); place second channel at "eyes of the knee" lateral and medial to patella tendon just above tibia.

